Uro-Tainer®

Suby G B. Braun

BG	Указания за употреба	2
CZ	Informace pro použití	4
DE	Gebrauchsanweisung	6
DK	Brugsanvisning	8
ES	Instrucciones de uso	10
FI	Käyttöohje	12
FR	Mode d'emploi	14
GB	Information for use	16
HU	Használati utasítás	18
IT	Istruzioni per l'uso	20
NL	Gebruiksaanwijzing	22
NO	Brukerveiledning	24
PT	Instruções de uso	26
SE	Bruksanvisning	28
SK	Informácie pre použitie	30

Approval for Printing

B|BRAUN Melsungen AG

Approved for Printing ☐

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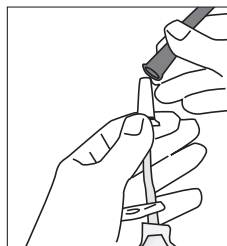
Uro-Tainer® Suby G B. Braun

Maintenance solution for urethral
and suprapubic catheters
Sterile, free of endotoxins

Composition

100 ml solution contains:

Citric monohydrate acid	3.23 g
Light magnesium oxide	0.38 g
Sodium bicarbonate	0.7 g
Disodium edetate 2H ₂ O (excipient) in Water for Injections	0.01 g

Description

A citric acid-based solution for indwelling urethra and suprapubic catheters to dissolve encrustation that could lead to the obstruction of the catheter.

Uro-Tainer® is a ready-to-use, disposable system. It consists of a PVC sachet, in a protective plastic overwrapping, containing 50 ml or 100 ml of sterile solution, a flexible tube fitted with a clamp, and a sterile universal connector protected by a plastic cap with a tamperproof seal.

Dosage

Unless otherwise advised by a doctor, depending on the seriousness of the patient's condition, a 5 min instillation once to twice per day to once per week.

Overdosage

No clinical data exists concerning overdosage.

Important information

- This solution is intended for urinary catheter rinsing only : not for intravenous infusion.
- Instil the solution by gravity feed, avoid force.
- Precautions should be taken when performing bladder washout on patients with spinal injuries because of the risk of autonomic dysreflexia.
- Use after urinary tract surgery should be evaluated by a physician.
- Not to be used in cases of cystitis or other uro-genital conditions that can produce haematuria (blood in the urine). Refer to a doctor who will decide on the treatment to be followed.
- *The solution is not a vehicle for additives.* It must not be used for the dissolution of medication.
- The acidity of the Suby G solution (pH ≈ 4) can alter the effect of other medication. This should be noted particularly in connection with the treatment of urinary infections with antibiotics.
- Apply using aseptic technique.
- Only to be used if the overwrapping, the contents, and the sealing cap are intact, and the solution is clear.

- Dispose of any partially used sachet and any unused solution.
- Store out of the reach of children.
- Refer to the operating diagram on the next page.
- This is a single-use system. After opening, it must be used and disposed of promptly.
- Discard properly after use.
- Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.
- Animal experiments have shown DEHP to be potentially toxic to reproduction. Given the current state of scientific knowledge, a risk (in case of long-term use) especially for premature male babies cannot be completely excluded. As a precaution, the application of medical products containing DEHP should be restricted to short-term use for pregnant women, breastfeeding mothers, infants, and children.

Secondary effects

- Citric acid may cause some patients to experience slight irritation and even temporary pain, a burning sensation, or spasms of the bladder (desire to urinate). In these cases the application should be carried out less frequently, or alternated with an application of NaCl 0.9%. In general, however, this treatment is well tolerated and these secondary effects do not require it to be interrupted.
- If the liquid exits from the urethra during the irrigation, the catheter may no longer be in the bladder and the patient may require a new catheter.
- If the liquid does not flow, the catheter may be kinked in the bladder or it may be blocked and may require changing.

Expiry date

Do not use after the date marked «EXP» on the back of the sachet.

Storage

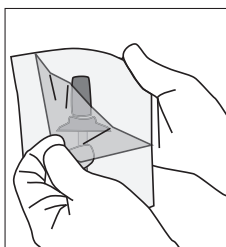
Store in the original packaging, protected from light, at a temperature not exceeding 25 °C.

Latest Information update

06/2010

The Uro-Tainer® sachet is supplied sterile and should not be removed from its overwrapping until required for use. If desired, bring solution to body temperature by immersing the wrapped sachet in warm water.

1. Tear open overwrapping and remove Uro-Tainer® sachet.


2. Close Uro-Tainer® tube with plastic clamp.


3. Remove tamper proof seal.

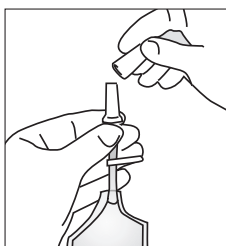4. Twist and withdraw cap without touching sterile connector.

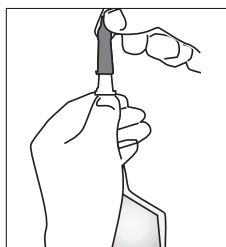5. Insert connector into the catheter funnel.


6. Holding the Uro-Tainer® above the level of the bladder, release the clamp to allow the fluid to drain in by gravity.

7. After 5 minutes, lower the Uro-Tainer® below the level of the bladder to drain fluid back into the bag.

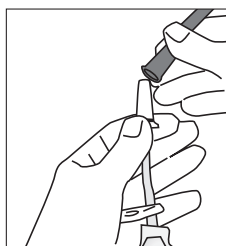8. Clamp tube and remove the connector from the catheter by turning gently. Connect the catheter to a sterile drainage bag. Dispose of Uro-Tainer® sachet and wash hands.

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B. Braun Medical AG